

Medtronic

Did you know

you can temporarily change your **SmartGuard™** feature target?



The SmartGuard™ technology targets are:

100 mg/dL (default)

110 mg/dL

120 mg/dL

The target can be temporarily changed to 150 mg/dL at times when less insulin is needed.

Examples of when to use Temp Target:



Exercise



Higher activity
(Yard work, cleaning, painting)



Help prevent lows

When using SmartGuard™ technology, the Temp Target can be accessed from the SmartGuard™ menu. The duration can be set from 30 minutes up to 24 hours each time.

Best Practice:

Set the temp target 1-2 hours before activity, keep it on during, and stop at the end.

Notes:

- Auto corrections are not delivered during a Temp Target.
- After the programmed time, the Temp Target will automatically end, and the programmed SmartGuard™ target will be used. The Temp Target can also be canceled at any time.

Always seek the advice of your healthcare provider with any questions you may have on your medical condition or treatment.

Important Safety Information for MiniMed™ 780G System:

MiniMed™ 780G System is for type 1 ages 7 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For details, please visit <https://www.medtronicdiabetes.com/support/download-library/user-guides> and www.medtronicdiabetes.com/important-safety-information for complete details.

US-IPT-2300624 ©2023 Medtronic. All rights reserved. Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.