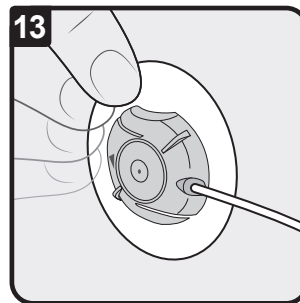
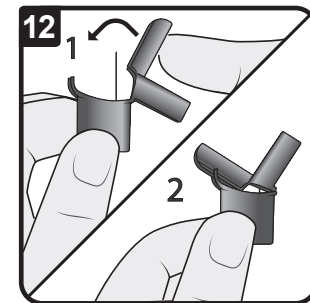
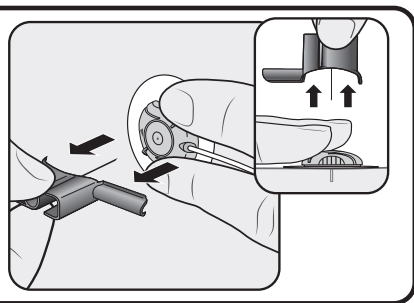
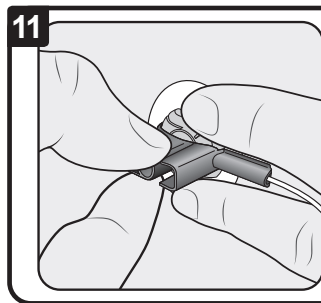
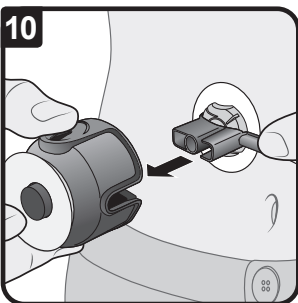
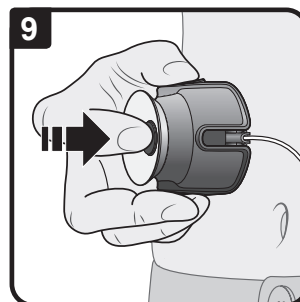
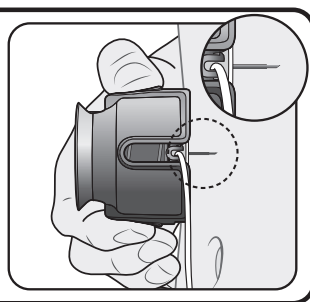
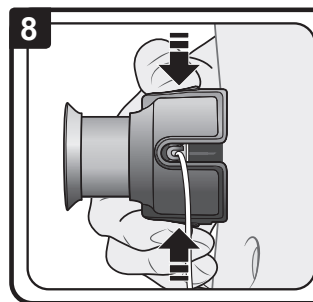
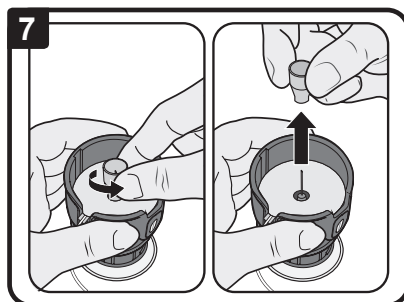
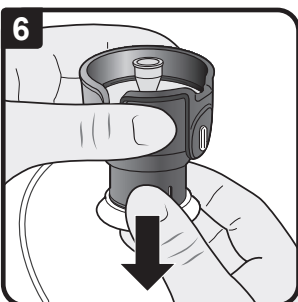
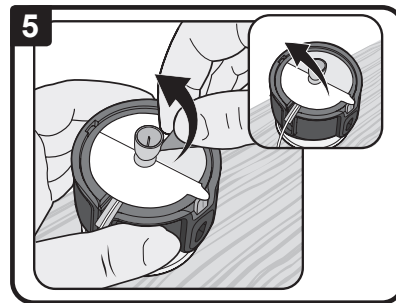
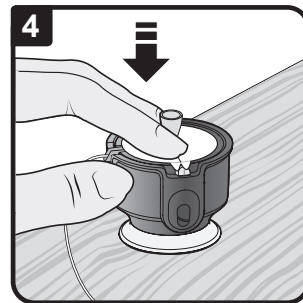
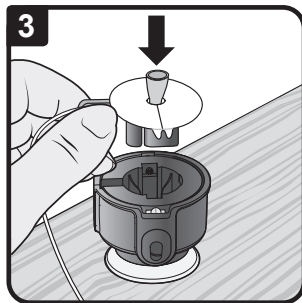
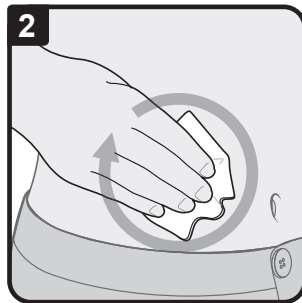
MiniMed™ Quick-serter™

MMT-305QS

User Guide • Guide d'utilisation • Gebrauchsanweisung •
Guía del usuario • Gebruiksaanwijzing • Guida per
l'utente • Bruksanvisning • Käyttöopas • Bruksanvisning •
Brugsanvisning • Guia do utilizador • Navodila za uporabo •
Uživatelská příručka • Használati útmutató • Podręcznik
użytkownika • Kullanıcı Kılavuzu • Používateľská príručka •
Οδηγός χρήστη • Руководство пользователя • 用户指南 •
Guia do usuário •

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Indications for Use

The MiniMed™ Quick-serter™ is indicated to use as an aid for inserting compatible devices. It is for use by a single patient. It is not for use by multiple patients.

Contraindications

Do not use the Quick-serter with non-compatible devices. Medtronic MiniMed cannot guarantee the safety or efficacy of the Quick-serter if non-compatible devices are used.

Warnings

Failure to follow instructions may result in improper insertion, pain, or injury.

Always inspect the packaging of the infusion set for damage. Do not use the infusion set if there is damage.

If the infusion set is not securely placed in the Quick-serter, improper insertion, pain, or minor injury may occur.

Never point the loaded Quick-serter toward any body part where insertion is not desired. This is to avoid an accidental needle stick.

Carefully remove the needle guard before inserting the infusion set into your skin. The needle should be straight and point outward. Do not use the infusion set if the needle is bent.

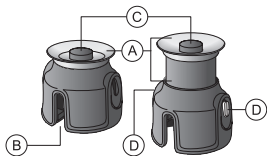
Precautions

You should perform your first set-up of the Quick-serter in the presence of your healthcare professional.

Warranty

For product warranty information, please contact your local help line or representative.

Components

- (A) Handle
- (B) Tubing slot
- (C) Release button
- (D) Side buttons

Instructions

Unfold the front cover of this booklet to view images that are numbered to the steps below. Read all instructions carefully before using the Quick-serter. For information about pump therapy, potential errors, and possible risks and their causes, consult your pump user guide before connecting the infusion set. Refer to your infusion set user guide before using the Quick-serter. If you are using the Quick-serter for the first time, you should perform your first set-up in the presence of your healthcare professional.

Preparation

1. Wash your hands with soap and water.
2. Clean the insertion site. Use an alcohol wipe or other disinfectant as directed by your healthcare professional. The area must be dry before you insert the infusion set.

Using the Quick-serter

3. Place the Quick-serter onto a sturdy flat surface with the handle facing downward. Place the infusion set into the Quick-serter. If needed, place infusion set tubing in the tubing slot.
4. Seat the infusion set securely within the Quick-serter. To do this, place your fingers on the adhesive liner and push the infusion set down.
5. Remove both pieces of the adhesive liner.

6. Pick up the Quick-serter and pull the handle down until it clicks.
7. Twist the needle guard until loose. Carefully remove the needle guard. Do not touch or bend the needle.
8. Place the Quick-serter at the desired infusion site. Then, press the two side buttons at the same time to insert the infusion set.
9. Press the release button on top of the Quick-serter to release the infusion set.
10. Hold the infusion set tubing with one hand. Then, remove the Quick-serter by lifting it away from your body with your other hand.
11. Pull the needle housing away from the infusion set with one hand. Gently press the tape against your skin with your other hand.
12. Close the needle housing. Then, discard the shielded needle in a sharps container.
13. Smooth out the tape securely against your skin.

Cleaning

The MiniMed Quick-serter is intended for single-patient use. Always clean the serter after each use.

To clean the serter, you will need the following materials: mild liquid soap (for example, Ivory® liquid soap), a soft-bristled toddler toothbrush, and a container.

To clean the serter:

1. Rinse the serter under running room temperature tap water for 30 seconds. Make sure all hard-to-reach areas are rinsed completely.
2. Prepare a mild liquid soap solution using one teaspoon (five milliliters) of mild liquid soap with one gallon (16 cups or 3.8 liters) of room temperature tap water.
3. Fully submerge the serter in the mild liquid soap solution and soak it for one minute.

4. Thoroughly brush the entire surface using a soft-bristled toddler toothbrush. Pay close attention to brushing hard-to-clean areas until visibly clean.
5. Rinse the serter with running room temperature tap water for at least 30 seconds and until all visible liquid soap is gone.
6. Thoroughly inspect the insertion device. If any dirt is visible, then repeat steps 3 to 5.
7. Shake off any excess water or moisture, and place the serter on a clean, dry cloth to air dry.

Store serter in the released position to maintain optimum product performance and life.

©2015 Medtronic MiniMed, Inc. All rights reserved.

MiniMed™ and Quick-serter™ are trademarks of Medtronic MiniMed, Inc.

Ivory® is a registered trademark of The Procter & Gamble Company.

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Medtronic B.V.