

Medtronic infusion sets

A set for every body. A set for every lifestyle.

	MiniMed™ Mio™ Advance	MiniMed™ Mio™	MiniMed™ Quick-Set™	MiniMed™ Silhouette™	MiniMed™ Sure-T™
					
Infusion set design	<ul style="list-style-type: none"> • Dexterity issues • Needle anxiety 	<ul style="list-style-type: none"> • Lean/active • Alternative site usage 	<ul style="list-style-type: none"> • Alternative site usage • Dexterity issues 	<ul style="list-style-type: none"> • Lean/active • Alternative site usage 	<ul style="list-style-type: none"> • History of bent cannula • History of scar tissue • Teflon allergy
Recommended body type	Medium/High BMI*	Low BMI	Medium/High BMI	Low BMI	Medium/High BMI
Insertion device	All in one	All in one	Quick-serter™ or Manual insertion	MiniMed™ Sil-serter™ or Manual insertion	Manual insertion
Cannula/needle type	Soft	Soft	Soft	Soft	Steel
Insertion angle	90°	90°	90°	30°- 45°	90°
Cannula length	6 mm, 9 mm	6 mm, 9 mm	6 mm, 9 mm	13 mm, 17 mm	6 mm, 8 mm, 10 mm
Tubing length	23 in, 43 in	23 in, 32 in	18 in, 23 in, 32 in, 43 in	18 in, 23 in, 32 in, 43 in	18 in, 23 in, 32 in
Cannula thickness (gauge)	25G	25G	25G	25G	29G
Change frequency	2-3 days	2-3 days	2-3 days	2-3 days	1-2 days

*Body Mass Index

Important Safety Information: Infusion sets are indicated for the subcutaneous infusion of insulin from an infusion pump. Infusion sets are indicated for subcutaneous use only and not for intravenous (IV) infusion or the infusion of blood or blood products. Inaccurate medication delivery, infection and/or site irritation may result from improper insertion and maintenance of the infusion site. Before insertion, clean the insertion site with isopropyl alcohol. Remove the needle guard before inserting the infusion set. If using this infusion set for the first time, do the first set-up in the presence of your healthcare professional. Do not leave air in the infusion set. Prime completely. Check frequently to make sure the soft cannula remains firmly in place as you may not feel pain if it pulls out. The soft cannula must always be completely inserted to receive the full amount of medication. If the infusion site becomes inflamed, replace the set, and use a new site until the first site has healed. Replace the infusion set if the tape becomes loose, or if the soft cannula becomes fully or partially dislodged from the skin. Regularly replace the infusion set as indicated in the instructions for use, or per the insulin labeling, whichever duration is shorter.

US-DBA-2200073 ©2022 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company.

Medtronic