


Medtronic

How to lend a hand with a sensor arm insertion

Does your loved one with diabetes need a hand with inserting their sensor?

Here are a few tips for how you can help support and make them feel like a champion!

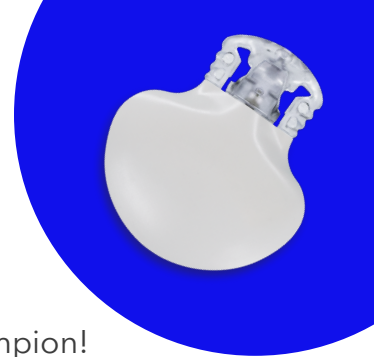

Use protective barriers:

Wear latex gloves to ensure a clean process.



Stabilize the sensor:

After the sensor is inserted and the inserter is removed, hold the base of the sensor firmly to prevent movement.



Remove the needle:

Using a piece of gauze, hold the needle housing and pull straight out. The needle will retract into the housing.



Flatten the adhesive pad:

Make sure the adhesive pad lies flat against the skin.



Apply the oval tape:

Place the wide part of the Oval Tape over the round part of the sensor.



Attach the transmitter:

Click the transmitter into place and cover it with the thin white adhesive strip.



Apply the second oval tape:

Apply a second piece of Oval Tape in the opposite direction of the first.



Additional coverage:

You may be asked to place an extra piece of tape over the entire site for added security or for added coverage.



Encourage and support:

Give your loved one a pat on the back. Living with diabetes and asking for help isn't always easy!

Approved for the back of the upper arm for ages 7 and older.

Guardian™ 4 sensor approved for the back of the upper arm for ages 7 and older.

Guardian™ Sensor 3 approved for the back of the arm and abdomen for ages 14+ and abdomen and upper buttock for ages 7 to 13.



For additional resources check out this video
bit.ly/InsertingG4S



To shop for tape and prep wipes visit
<https://www.diabetes.shop/tapes--prep-wipes>



Need more help? Scan the QR code or visit
medtronicdiabetes.com/support
to explore all resources that Medtronic provides.



Have more questions?
Call Tech Support at 1-800-646-4633, option 1.
24 hours a day, 365 days per year.



Consult with healthcare provider about treatment decisions.

Important safety information: MiniMed™ 780G system

MiniMed™ 780G System is for type 1 ages 7 and over. Prescription required. **WARNING: Do not use SmartGuard™ feature for people who require less than 8 units or more than 250 units of insulin/day.** For details, please visit <https://www.medtronicdiabetes.com/download-library> and www.medtronicdiabetes.com/important-safety-information for complete details.

US-CGM-2400060 © 2024 Medtronic. All rights reserved. Medtronic, Medtronic logo and Engineering the extraordinary are trademarks of Medtronic. Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

Medtronic